

Test Name	Test Code	Collection Container	Patient Instructions	Pour-off Container(s)	Special Notes
Urinalysis (UA): UA, Complete UA, Macroscopic (dipstick) UA, w/Reflex to microscopic UA, Microscopic UA, Screen	5463 6448 7909 8563 7048		Midstream Allow a small amount of urine to pass into toilet. Midway through urination, place container in the flow of urine to catch a sample. Finish voiding in toilet.		Transfer urine from pediatric bags to a clean container, then to pour-off container, dispose of the bag while following hygiene guidelines.
Urine Culture (UCx): UCx (Routine)	395		Clean Catch Midstream Wash hands. Use one (1) soap towelette, then three (3) saline towelettes. Follow midstream instructions listed above.		No additional tests can be conducted using this tube.
Urinalysis AND Urine Culture: UA, Complete w/ Reflex to Culture Culture & Sensitivity	3020		Clean Catch Midstream Wash hands. Use one (1) soap towelette, then three (3) saline towelettes. Follow midstream instructions listed above.		For pediatric and renal abnormality patient samples, under 5 mL, please send a refrigerated specimen in a sterile cup labeled "Pediatric and Renal Abnormalities."
Chlamydia trachomatis RNA, TMA Neisseria gonorrhoeae RNA, TMA, Chlamydia/N. Gonorrhoeae/TMA	11361 11362 11363		First-Catch Should not urinate within 1 hour before collecting sample. Collect first passing of urine into container provided. 20-30 mls of urine is needed to test.		Using the provided pipette, add urine until the liquid level is between the two black lines
Albumin, Random Urine w/Creatinine Osmolality Urine Random Urine w/Creatinine Protein, Total, Random Urine w/Creatinine UDS-HOSPITAL Protein, Total, 24-hr Urine w/Creatinine Electrophoresis & Total Protein, Random Urine hCG, Qualitative, Urine	6517 678 8459 1715 59084 757 8525 396		Random Urine Collection Hold container a few inches from urethra and urinate until container is half full. Finish voiding in toilet. Replace lid and screw tightly.	 *Keep in container	Cover the container with Parafilm lid to prevent leakage during transport. The specimen volume must remain between 1mL (min) and 5mL (max).
STI Panel, QL, PCR Vaginosis/Vaginitis Panel, QL, PCR Candidiasis Panel, Qual, PCR Women's Health Panel, Qual, PCR	97641 97712 97642 97711		Random Urine Collection Hold container a few inches from urethra and urinate until container is half full. Finish voiding in toilet. Replace lid and screw tightly.	 *Keep in container	Cover the container with Parafilm lid to prevent leakage during transport. The specimen volume must remain between 1mL (min) and 5mL (max).

*** PLEASE NOTE: Urine samples sent to the laboratory that are more than 5 mL and not in the correct pour-off container will be rejected, leading to a recollection error.***

*Draw a line on collection container and instruct patient to notify if unable to provide quantity.

**DLO accepts refrigerated specimen for UA on pediatric patients